

Two white, cylindrical wax sticks are shown against a black background. They are positioned diagonally, one slightly above and to the right of the other. The sticks have a smooth, slightly textured surface and rounded ends.

AEGIS LIFESCIENCES
Protection Through Perfection

ISO 13485 & GMP CERTIFIED
215/216, Mahagujarat Estate, Gam : Moraiya,
Tal. Sanand, Dist.: Ahmedabad-382213.
Fax: +91 2717 294295, Ph.: +91 2717 294636
Email: sales@aegis-lifesciences.com
Web.: www.aegis-lifesciences.com

- Quick haemostasis in anorectal surgeries
- Best suitable for post operative haemostasis
- Painless and easy tampon discharge
- High blood absorption capacity
- Surgispon® Tampon is provided with an opening into which a flatus drain can be introduced

Anal Tampon

Benefits/advantage

Surgispon® anal reduces post-operative pains more effectively than other known forms of anal tampons.

- High absorption capacity - up to 50 times of its own weight.
- Surgispon® anal gelatine sponge liquefies rapidly and is spontaneously expelled within 1 to 2 days.
- Used as a carrier for drugs like antibiotics, thrombin and haemostats glue.
- The gelatin sponge application can save up to 4 days of hospital stay for each rectal patient.
- Significantly increases hemostasis without increasing pain and discomfort.
- The immediate insertion of an absorbable hemostatic gelatin sponge can reduce the frequency of rectal bleeding.

Features

- Non-toxic
- Non allergenic
- Non-immunogenic
- Non-pyrogenic
- Biocompatible
- It has a non-water-soluble property
- Attaches to the bleeding tissue
- Used dry or soaked with physiological solution well adhering ability to the bleeding site
- Implantable and completely resorbable
- Does not stick to instruments

Surgispon® is manufactured from highly purified & pH neutral pharmaceutical gelatin foam of uniform porosity. Surgispon® is single-use & gamma sterilised absorbable haemostatic gelatin sponge. Surgispon® anal is used in haemorrhoid operations and other surgical procedures in the anal, vaginal and rectal regions.

Painless and easy tampon discharge. The surgispon® is easy expelled at first bowel movement, requiring no additional nursing intervention.

Where to use

- Anorectal surgery – suitable to better facilitate post operative haemostasis.
- To achieve fast hemostasis in
 - ▲ Haemorrhoidectomies.
 - ▲ Anal fistula and fissures.
 - ▲ Curettage and Hysterectomies.
 - ▲ Surgery for rectal cancer.
 - ▲ Rectal prolapse.
 - ▲ Colorectal surgery.
 - ▲ Insertion into rectum after transrectal prostate needle biopsy, for haemostasis

How to use

• For rectal application

In anal and rectal surgery, introduce a dry anal tampon with the help of a proctoscope at the end of the surgical procedure. Place dry gauze in the rima natis. The buttocks must be held together by means of a T bandage.

Bacterial invasion ensures the rapid dissolution of the sponge which softens and is spontaneously removed by defecation.

The Surgispon® Tampon is provided with an opening into which a flatus drain can be introduced.



After performing excision of hemorrhoidal piles, the operation is terminated by placing hemostatic absorbable gelatin sponge tampon in the anal canal.

• For Vaginal application

Shape Surgispon® Tampon to fit speculum. Insert, by means of a speculum a dry or soaked Surgispon® Tampon

Surgispon® Tampon may be used in combination with antibiotics, thrombin, chemo therapeutics without reduction of the haemostatic effect. Carefully read applicable instructions for use before usage.

'Biological Safety & Performance evaluation' has been performed on Surgispon® according to ISO 10993 and USP standards Surgispon® has passed all applicable tests. A summary of these studies can be obtained on demand.

